

## Planning to Pump at NeoCon? Let MilkMate make it easier for you.

### Meet MilkMate:

The revolutionary breast pumping solution built for the workplace. If you plan to continue pumping at the conference, MilkMate is here to make this transition as smooth as possible for you.

### What is included with MilkMate?

- ✓ Multi-user, high performance breast pump
- ✓ Preassembled, presterilized parts that come ready for immediate use.
- ✓ A user-friendly app and extensive user guide for easy use and navigation.



### Ready to get pumping? Here's how to get started:

**Step 1:** Leave the gear at home!

**Step 2:** Download the MilkMate app.

**Step 3:** Select a preassembled and presterilized kit and begin pumping!

- **Note:** MilkMate provides size 27 breast shields. If you need a smaller size, MilkMate provides reusable inserts to allow for size 15, 17 and 21 shields, to be used in conjunction with our size 27 shields. If you need a different size, please reach out to MilkMate at [support@milkmate.com](mailto:support@milkmate.com).

**Step 4:** Each time you use a breast shield kit, scan the provided QR code on the MilkMate station so that the MilkMate team can ensure your station never runs out of inventory!

**Step 5:** After each session, recycle your used breast shield kit.

- **Note:** MilkMate's single-use breast shield kits are fully recyclable, and each station should come equipped with a recycling bin for ease of use

**Step 6:** Place your pumped milk in the refrigerator. Don't forget to bring it home at the end of the day!

- **Note:** MilkMate does recommend that you bring your own cooler bag to ensure your milk stays cold on your commute home from work

### Need more information?

[Instagram](#) | [Blog](#) | [Introductory Video](#) | [FAQ](#) | Email: [support@milkmate.com](mailto:support@milkmate.com)